## **ESPAÑOL**

### **MONTURAS DE GAFAS GRADUADAS**

#### **CAMPO DE USO**

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### **OPTICAL FRAMES**

#### **FIELD OF USE**

Prescription glasses frames are medical devices whose main use is the association with ophthalmic lenses prescribed for the correction of visual defects. Prescription glasses frames meet the requirements of the regulations of Regulation (EU) 2017/745.

#### **Warnings**

It cannot be excluded that particularly sensitive or allergic subjects may manifest adverse reactions due to contact with the materials and treatments used for the frames. The mount comes with presentation lenses not intended for use, but only to maintain the shape of the glasses and offer a test of the mount closer to the final reality. The assembly of diopter ophthalmic lenses is reserved exclusively for qualified professional operators. Any other use is prohibited. The user is obliged to inform the manufacturer and the national competent authority of any serious incident that may occur in relation to this medical device.

#### **CONSERVATION AND CLEANING**

Prescription glasses frames should be stored at a temperature between -10°C and +35°C in their cover. Avoid sun exposure in enclosed and un-conditioned environments (e.g., the car dashboard), as temperatures reached in these conditions can damage the functionality of the mount. In case of occasional contact with agents such as salt water (seawater), pool water, chemicals such as cosmetics, hair lac seals, sunscreens and insect repellents, immediate cleaning with a damp cloth and a normal neutral detergent is recommended, and drying with a soft, clean cloth. Avoid the use of solvents (e.g. alcohol and acetone) or composites containing them and aggressive chemical detergents that may compromise the functionality of glasses. Do not use the product in case it is visibly damaged. Dispose of the product responsibly; remove the product in accordance with current local regulations.

#### **marked**

The model identifier is located on the inside of the glasses. The date of production is shown on the label of the individual packaging of the glasses.

#### **NOTIFICATIONS/CONTACTS**

In case it is not possible to contact the point of sale where the product was purchased, for claims or anomalies, direct the request to: [central@superb-vision.eu](mailto:central@superb-vision.eu)

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CO., LTD**